**vivamel**  
contact

**vivamel**  
contact

**vivamel**  
contact

**Navodilo za uporabo**  
**VIVAMEL CONTACT - KONTAKTNA MREŽICA Z MEDICINSKIM MEDOM**

Pred prvo uporabo izdelka Vivamel Contact – Kontaktna mrežica z medicinskim medom natančno preberite navodilo. Navodilo shranite. Morda ga boste želeli ponovno prebrati.

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Pomen simbolov:

**Instructions for use**  
**VIVAMEL CONTACT – CONTACT MESH WITH MEDICAL HONEY**

Before first use of Vivamel Contact – Contact mesh with medical honey read the instructions carefully. Keep the instructions for future use.

**WHAT IS VIVAMEL CONTACT - CONTACT MESH WITH MEDICAL HONEY?**

Vivamel Contact – Contact mesh with medical honey is intended for treating wounds with weak secretion and protecting the wound bed. The mesh is soft, thin, and covered with medical honey. Due to its extreme flexibility, it provides improved contact with the wound bed, while honey prevents the dressing from sticking onto the wound. Available in two sizes: 10 x 10 cm and 5 x 5 cm. The larger dressing (10 x 10 cm) is covered with 2 g and the smaller one (5 x 5 cm) with 0.5 g of medical honey.

**Medical honey**

Medical chestnut honey has a high sugar content (min. 80 %), a low water content (max. 17 %), and an acid pH value. Thanks to these physical properties, the osmotic activity of honey is very high. Osmotic pressure drives fluid out of the cells of microorganisms, inhibiting their growth and at the same time stimulating wound healing as it stimulates the flow of necessary substances in the body. Chestnut honey is a rich source of vitamins, enzymes, and minerals (especially calcium, manganese, and potassium) which additionally stimulate wound healing and shorten the healing period. Honey enables moist wound healing and creates mechanical protection on the wound.

**Contact mesh**

Contact mesh is made of polyester woven fabric. The mesh is not absorbent but its porous structure allows the exudate to flow from the wound through to a second dressing. Therefore, it is necessary to use a secondary absorbent dressing.

**ADVANTAGES OF USING VIVAMEL CONTACT – CONTACT MESH WITH MEDICAL HONEY AND ITS EFFECT:**

- enables moist wound healing
- lowers risk of wound infection
- reduces unpleasant odours
- stimulates necrotic tissue cleaning
- stimulates granulation and epithelisation
- promotes wound healing and prevents scarring
- no sticking onto the wound
- does not cause pain at dressing change
- easy to use
- no damage on the area surrounding the wound

**INTENDED USE OF VIVAMEL CONTACT – CONTACT MESH WITH MEDICAL HONEY**

Vivamel Contact – contact mesh with medical honey is designed for treating wound bed of a wide spectrum of wounds, from chronic to acute. It is particularly convenient for treating skin abrasions and scratches, wounds with weak secretion, and minor lesions. The mesh is not absorbent and therefore needs a secondary absorbent dressing or a plaster. When using the contact mesh, there is a possibility of only changing the secondary dressing, which provides protection to the wound bed and helps preserving healthy, newly formed tissue.

**HOW TO USE VIVAMEL CONTACT – CONTACT MESH WITH MEDICAL HONEY?**

1. First, rinse the wound with lukewarm running drinking water or saline.
2. Gently dry up the wound and its surrounding area.
3. Take the mesh with protective transparent foil out of the sterile pouch. Remove the foil from one side of the mesh and place the mesh onto the wound, sterile side with no foil on the wound. When placing it, make sure that the mesh covers the whole wound. It can be cut or fold and it may stretch over the wound margins.
4. Remove the protective foil from the other side of the mesh.
5. Protect the mesh by placing a secondary absorbent dressing of your choice over it. For wounds with a smaller amount of secretion, nonwoven compresses or plasters may be used. For wound with larger amounts of secretion, use a highly absorbent dressing or polyurethane foam. Fasten the compress, the absorbent dressing, or the polyurethane foam securely with a fastening bandage or adhesive tape, preventing the mesh from moving.

For easier application of Vivamel Contact – contact mesh with medical honey use sterile medical devices, e.g. sterile tweezers or tongs. The mesh can be cut in smaller pieces with sterile scissors while still protected by the protective transparent foil.

In case of black necrosis, surgical removal (debridement) is required, as necrosis arrests and slows down the healing process.

**HOW OFTEN SHOULD ONE CHANGE VIVAMEL CONTACT - CONTACT MESH WITH MEDICAL HONEY?**

At the beginning of the healing process, it is recommended that the mesh be changed every two days, in case of a larger amount of the secretion even every day, for the effects of honey are decreased by dissolving in wound secretion. Wounds with very little secretion require changing only twice a week. In case of mesh drying and sticking to the wound, moist it with saline solution. Remove a used mesh and dispose of it as infected waste. Clean and dry the wound and apply a new mesh as described above.

**POSSIBLE SIDE EFFECTS**

Due to honey's high osmotic activity, which is based on driving water molecules out of the wound, a burning sensation or a slight pain can occur after applying the mesh on the wound, but it usually goes away after 30 minutes to 2 hours. In case of unbearable pain, stop using Vivamel Contact – contact mesh with medical honey. Should any unknown side effects occur during the use of the product, ask your doctor or pharmacist or consult the manufacturer or distributor.

**WARNINGS**

Do not use the contact mesh if you are allergic to honey or have already experienced a hypersensitivity reaction to honey in the past. Do not use the contact mesh if the protective pouch has been damaged or opened. Do not use the contact mesh if the welds of the packaging are not sealed.

Once the mesh has been opened, never sterilise and reuse it. Before and after the application of the mesh always wash your hands.

The mesh is not suitable for eating. If consumed, it can cause blood sugar level to rise in diabetes patients or a hypersensitivity reaction in persons with honey allergy.

Avoid contact with eyes and mucosa. In case of accidental contact with eyes or mucosa, rinse with abundant quantities of water. For single use only.

Never reuse a used mesh. A used mesh does not contain honey or only a small amount of it, which decreases its effect. Reusing a contact mesh can lead into contamination of the product or the wound, possibly even an infection.

One mesh should never be used on two patients, in order to prevent contamination of the product or the wound, possibly even an infection.

Clinical studies of wound healing with Vivamel medical honey in patients with diabetes show very positive results, however, the treatment must be performed under a physician's supervision. Shelf life of the mesh is three years from the production date. After the expiration date, do not use the mesh and dispose of it as normal waste.

After four weeks of therapy with Vivamel contact mesh, a doctor should review the treatment.

**STORING VIVAMEL CONTACT – CONTACT MESH WITH MEDICAL HONEY**

Store product in a dry place at temperatures between 5 and 35 °C horizontally, otherwise honey will start leaking from the dressing. Avoid exposing the product to direct sunlight.

**VIVAMEL CONTACT – CONTACT MESH WITH MEDICAL HONEY**  
Vivamel Contact – contact mesh with medical honey 10 x 10 cm; Art. no.: 1054440  
Vivamel Contact – contact mesh with medical honey 5 x 5 cm; Art. no.: 1054441

Manufacturer: Tosama d.o.o., Vir, Šaranovičeva cesta 35, 1230 Domžale, Slovenia; Issue date: March 2016  
**www.tosama.si**

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